





Pulsewave[®] Cuff Positioning Guide

Open the cuff's Velcro® tab and pull the flap through the metal clip to loosen the cuff. Do not pull the cuff through the clip beyond the stopper installed in the cuff.

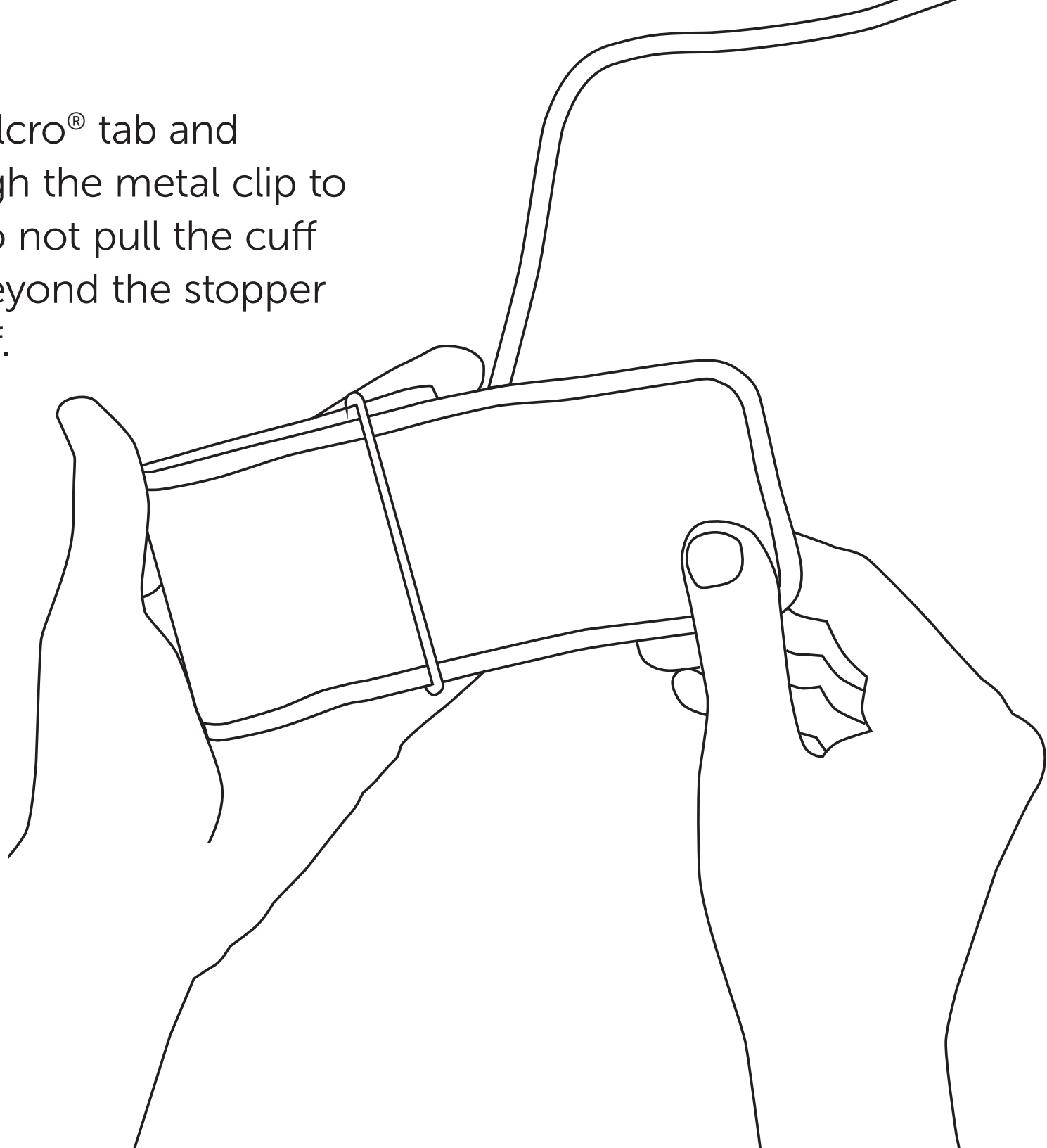


1

Remove any jewelry and roll up shirt sleeves. If possible, always place the cuff on your left wrist. Insert your hand through the cuff.

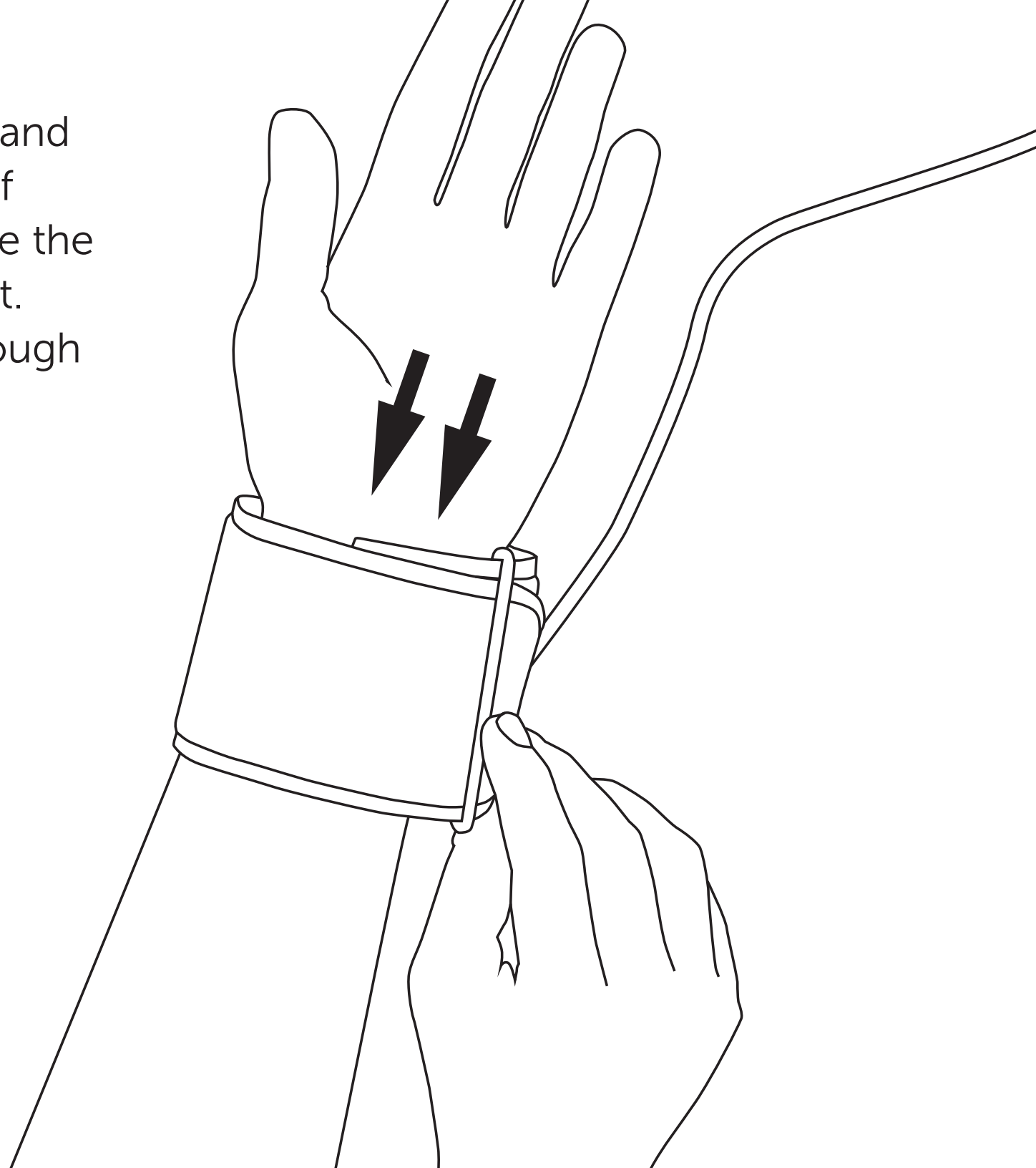


2

Pull the flap through the metal clip to tighten the cuff.

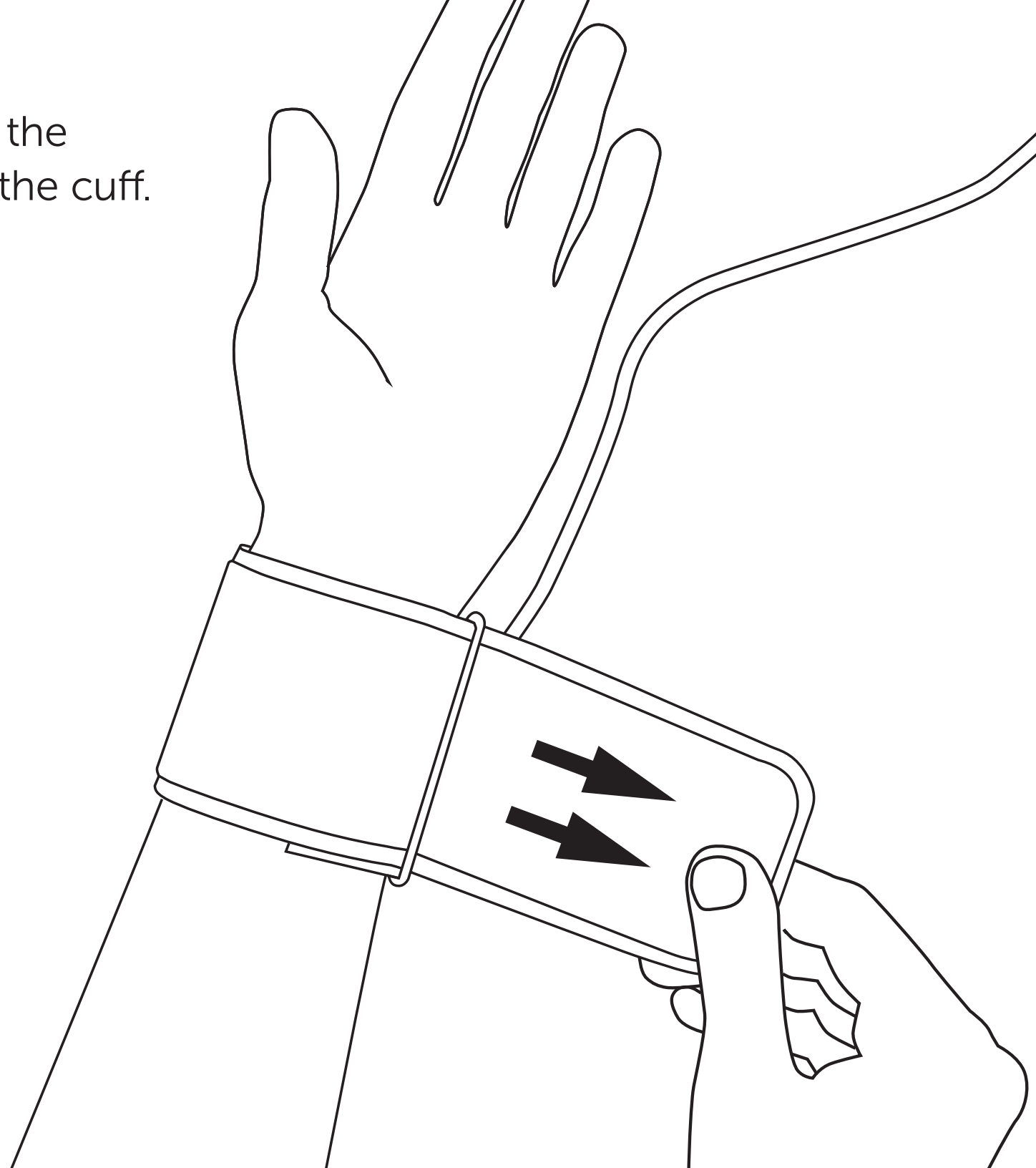


3

Bend the flap. Make sure the cuff is attached comfortably to your wrist. It should be snug, neither too tight nor too loose.

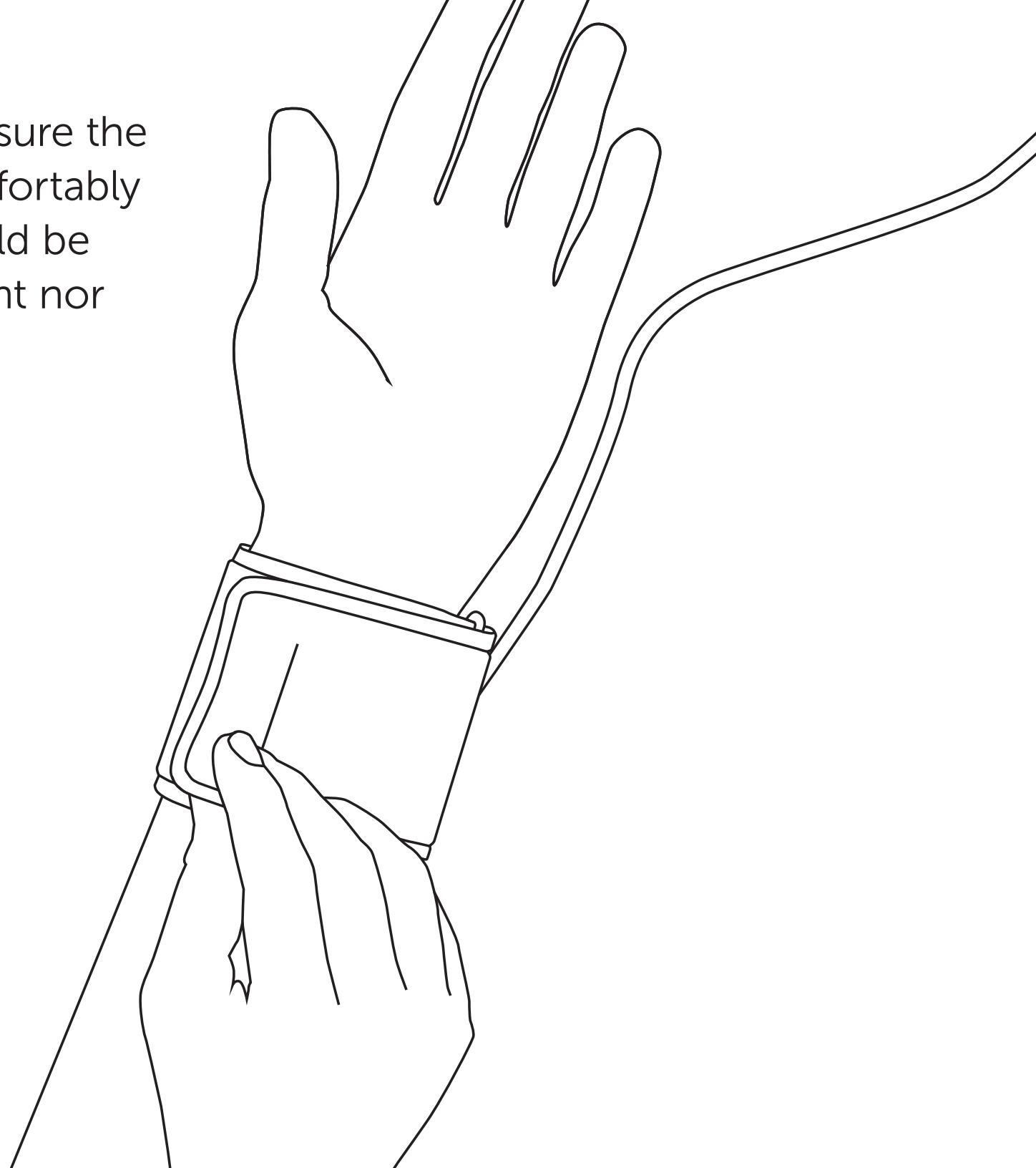

The inflatable cuff can be inverted or rotated as presented here for optimal comfort.

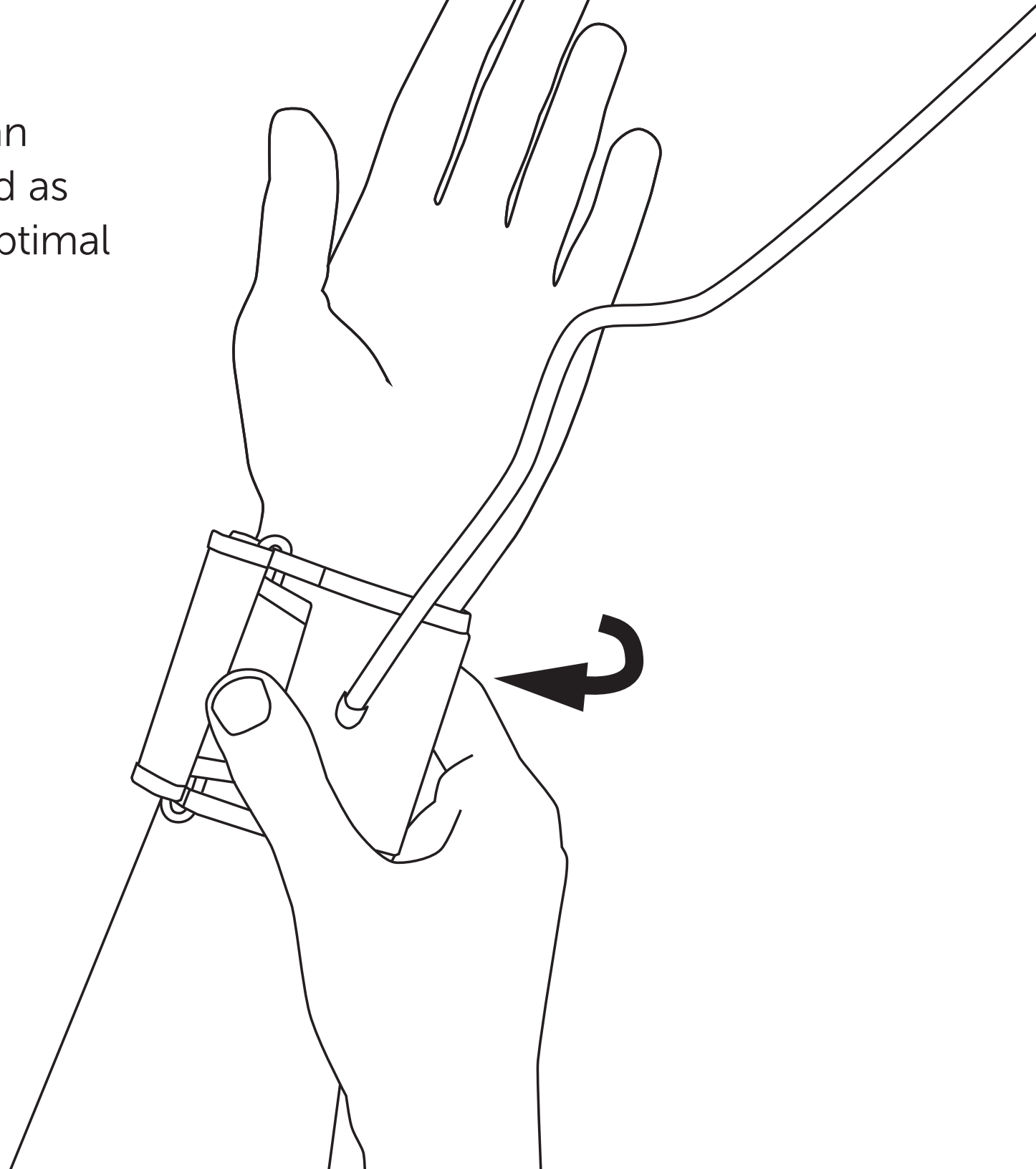



 Manufactured by:

Cloud DX Inc.

#100 - 72 Victoria Street South
Kitchener ON
N2G 4Y9
clouddx.com



European Representative:
Advena Ltd., Pure Offices
Plato Close, Warwick CV34 6WE UK

Customer Support:
1-888-543-0944 ext 1
support@clouddx.com

© 2016 Cloud DX Inc

Version 1.1

FDA
CLEARED

**HEALTH
CANADA**
LICENSED

